stimolatore
neuromuscolare

MODULO[®] medical 2-4

MODULO® medical 2-4

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Modulo medical 2 - 4 are the answer to an increase demand for an electrotherapy device easy to use and able to comply with different therapeutic requirements.

Modulo medical 2 - 4 are the first neuromuscular stimulators offering the operator the possibility, through SMART CARD and CABLE EQUIPMENT, to configure the same appliance for various operating solutions: **electrostimulation** (innervated and denervated muscle); **electroanalgesia** (low and medium frequency); **urological stimulation**; **electrodiagnosis**; **high voltage electrotherapy**; **microcurrent electrotherapy**; **ultrasound connection** for combined therapy.

Modulo medical 2 has two completely independent stimulation channels, electrically insulated from one another.

Modulo medical 4 has four completely independent stimulation channels, electrically insulated from one another.

Stimulation is controlled in current (CC) and voltage (CV). This feature is absolutely necessary for the connection to the ultrasound therapy devices of the Fisiosonic® family. The user can exploit different protocols per channel. Each protocol can consist of a sequence of up to 5 phases and each phase can use a different shape of wave.

Modulo medical 2 - 4 offer the operator the possibility, upon stimulation, of adjusting the current intensity of the single channel or of all of them at the same time.

Modulo medical 2 - 4 work plugged to the mains or with a battery.

They are devices simple to use equipped with a large graph display (240x128 pixel) allowing the operator to keep all functions under control.

A pleasant design, a user's interface equipped with a membrane keyboard and a display have a decidedly modern and professional look. The high current output (over 100 mA) makes

Modulo medical 2 - 4 two of the most powerful devices of their category.

Safety is guaranteed by the conformity with the Directive 93/42/EEC, modified by the Directive 2007/47/EC, and in the Directive 2004/108/EC.

MODULO®
medical 2-4

SMART CARDS

AVAILABLE CONFIGURATIONS: (SMART CARD + CABLE KIT)

- **CARD REHAB: stimulation of the normally innervated muscle:** balanced two-phase square wave, two-phase triangular wave, Kotz currents and physiological pulse. **Stimulation of the denervated muscle:** exponential single-phase and two-phase wave, single phase and two-phase triangular wave, partially denervated.

- **Antalgic currents:** Diadynamic (Single-phase, syncopated single-phase, diphase, syncopated diphase, short period and long period), Interferential (Bipolar, Isoplanar, Tetrapolar and Vectorial), AMF, TENS, rectangular wave and Spike (Continuous, Burst and Random), Faradic, (Rectangular, Triangular, Exponential, Neofaradic and Traebert).

- **Vehiculation currents:** Ionophoresis, Iontophoresis, basic Galvanic current and Continuous Interrupted.

- **Urological stimulation:** two-phase square wave, two-phase triangular wave, physiological pulse.

230 preset programs. 100 user's programs.

- **DIAGNOSIS CARD:** I./T curve innervated and denervated with calculation of chronaxie and reobasis. Square and exponential wave.

- **H.V. CARD: High voltage electrotherapy - single channel:** positive single-phase wave, negative single-phase (possibility of polarity inversion)

77 preset programs. 100 user's programs.

- **ULTRASOUND CARD:** Connection for ultrasound devices of the Fisiosonic® series for combined therapy.

100 preset programs. 100 user's programs.

- **MICROCURRENT CARD: Microcurrent electrotherapy (MET):** positive wave, negative wave, bipolar wave.

The microcurrent electrotherapy, indicated to relieve pain, uses a low level current (microampere) emitted with long impulses compared to the TENS current.

Unlike the latter, that have to be used in continuity to have prolonged effects, the microcurrents offer lasting and cumulative in time benefits.

6 preset programs. 100 user's programs.

- **PROFESSIONAL SPORTS CARD: made expressly for the sports professional:** balanced two-phase square wave with automatic variation of the stimulation frequency, twophase triangular wave, Kotz currents and physiological pulse.

181 preset programs. 100 user's programs.

Modulo medical 2 - 4 demand for their use a minimum equipment of a smart card and relevant kit, upon the client's choice.

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technical features:

- MODEL: MODULO medical 2 and MODULO medical 4.
- COMMERCIAL CLASSIFICATION: neuromuscular stimulator for electrotherapy.
- TECHNICAL CLASS: electromedical appliance, class II type BF.
- MEDICAL DEVICE CLASS: IIa (Dir. 93/42/EEC, modified by the Directive 2007/47/EC).
- REFERENCE STANDARDS: EN 60601-1 (IEC 60601-1), EN 60601-2-10 (IEC 60601-2-10), EN 60601-1-2 (IEC 60601-1-2), EN 60601-1-4 (IEC 60601-1-4).
- SIZE: 208x306x90 mm.
- WEIGHT: 1.5 Kg.
- POWER SUPPLY:
 - Power supply voltage 100, 240V.
 - Line frequency: 50-60 Hz.
 - Absorbed power: 24Watts.
- BATTERY POWER SUPPLY: 8,4V 3000mA/h Ni-MH (optional).
- STIMULATION CHANNELS:
 - MODULO medical 2: 2 completely independent and electrically insulated channels.
 - MODULO medical 4: 4 completely independent and electrically insulated channels.
- OUTPUT STAGE: controlled in CC current and CV voltage.
- CURRENT OUTPUT: 100mA max (in view of the type of current).
- DISPLAY: graphic display (240x128 pixel) backlight.
- TIMING: treatment duration setting 1 to 60 min.
- ACOUSTIC SIGNAL at the end of treatment.
- OPTIONAL ACCESSORIES:
 - SMART CARD + CABLE KIT
 - rechargeable battery 8.4V 3000mA/h Ni-MH
 - roller electrode
 - pencil electrode
 - single patient urological probes
 - trolley model DINAMICO
 - trolley model DUET
 - soft case
 - smart card holder.
- **CE** MARKING: the appliance complies with the requirements specified in the Directive 93/42/EEC, modified by the Directive 2007/47/EC, and in the Directive 2004/108/EC.

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FISIOLINE® instruments are covered by FONDIARIA-SAI product liability insurance.

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